



EN-Dynamic training means optimum exercising for everyone: men-women, elderly -youngsters, beginners-advanced.



After the automatic set-up with the EN-Track chip-card, the ideal body position can be set. In this correct posture the body is supported on essential points. Hereby the exercises are executed in an appropriate way: with correct posture and accurate load. Compensation is limited to a minimum. The units are easily accessible, easy adjustable and because of the EN-Track guiding unambiguous to use. Even individuals without training experience will feel very at ease in your centre.

**EN-DYNAMIC. FIT THE MEDICAL WAY:
IN THERAPY AND IN SPORTS**



- Adjustable from a seated position: the load is set by a simple push on a button
- Provides clear feedback to the user: the force is displayed digitally and the number of repetitions are counted automatically
- The large green letters are also visible for persons with visual limitations
- Manually adjustable
- With the chip-cards, the resistance and number of repetitions are set automatically
- Simple to operate and accurately adjustable
- No unnecessary high peak forces
- Silent during exercising
- Minimal increments are possible, thus accurate setting and smooth progression
- Wide range of load setting: from extremely light to very heavy
- Compact

EN-DYNAMIC TRUNK



1455922 EN-DYNAMIC LOW BACK TRAINER

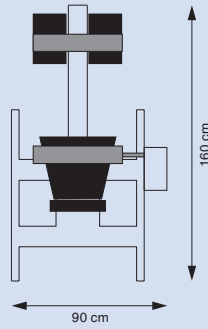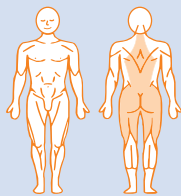


1455924 EN-DYNAMIC ABDOMINAL TRAINER

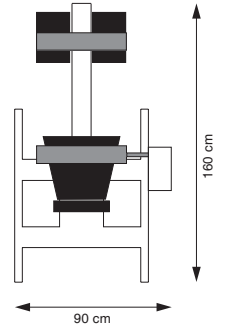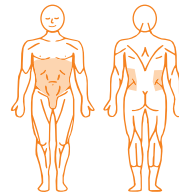


1455936 EN-DYNAMIC ROTATOR

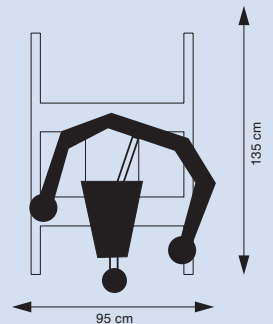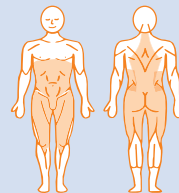


1455921 EN-DYNAMIC CHEST PRESS

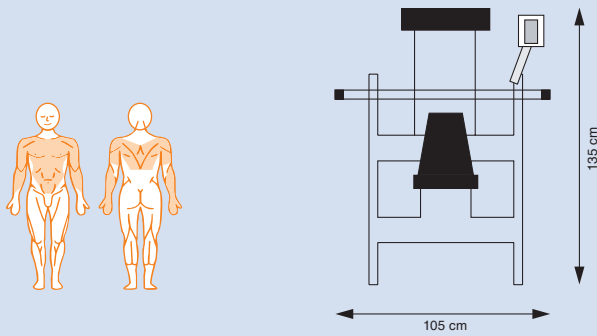

1455923 EN-DYNAMIC PULL DOWN


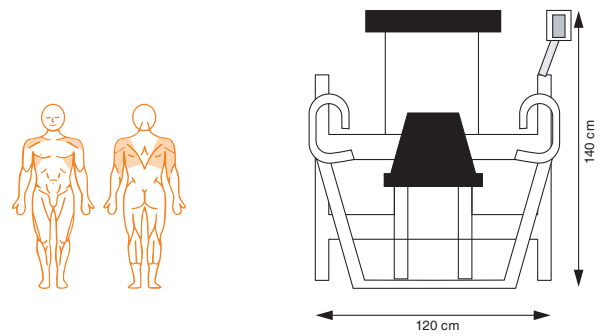


1455927 EN-DYNAMIC SHOULDER PRESS


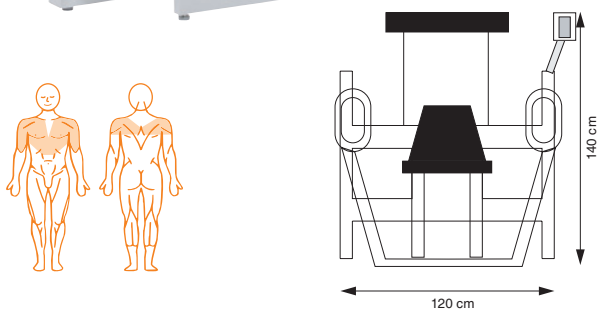


EN-DYNAMIC UPPER EXTREMITY



1455926 EN-DYNAMIC ROWING

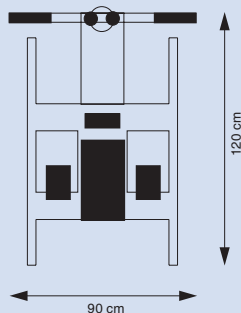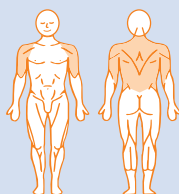


1455934 EN-DYNAMIC TRICEPS DIPS

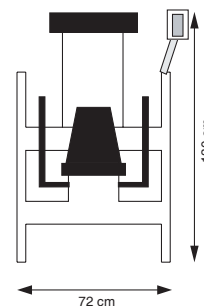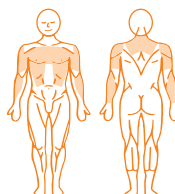


1455930 EN-DYNAMIC FLY

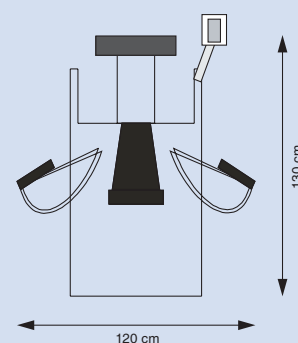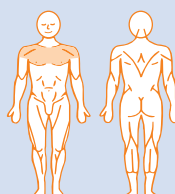


1455931 EN-DYNAMIC TOTAL HIP

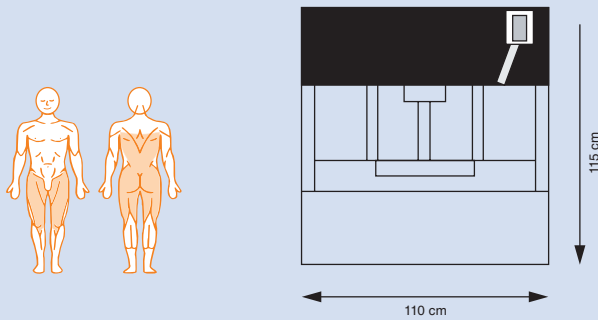


1455925 EN-DYNAMIC LEG EXTENSION

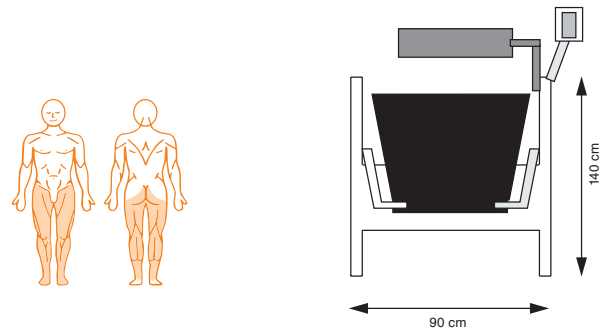

1455929 EN-DYNAMIC ABDUCTION


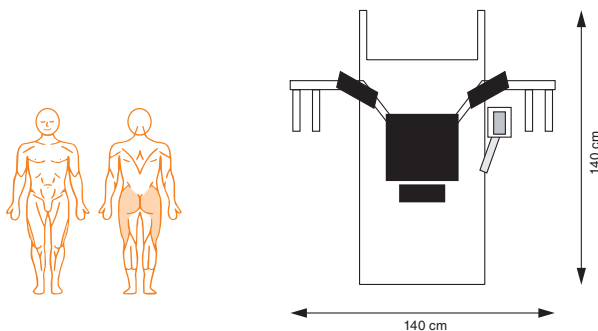


1455928 EN-DYNAMIC ADDUCTION


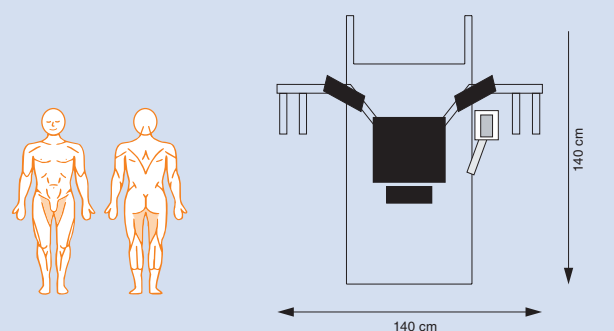


EN-DYNAMIC LOWER EXTREMITY



1455932 EN-DYNAMIC SEATED LEG CURL

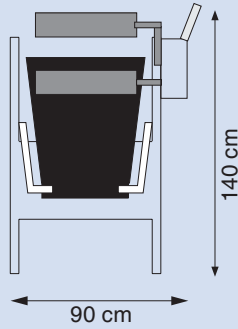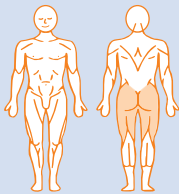

1455933 EN-DYNAMIC SEATED LEG PRESS

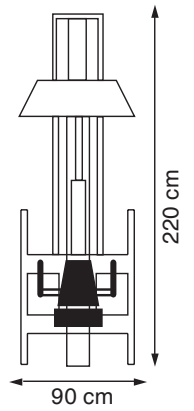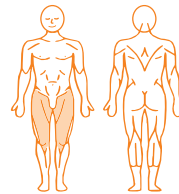

1455935 EN-DYNAMIC SQUAT

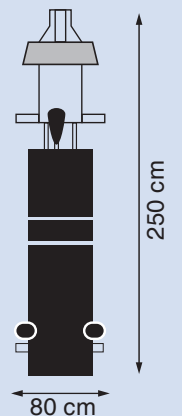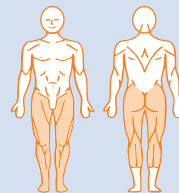


MDD OPTIONAL COLOURS

A matching EN-Dynamic for every interior. As in other practices, a pleasant ambience is important in physiotherapy practices. Clients must feel comfortable in the room where they are being treated. As each practice is furnished differently and has its own colour scheme, we regard it as important for the EN-Dynamic to fit in with your look and feel. That is why we can manufacture the EN-Dynamic in several upholstery colours.


UPHOLSTERY

Colour	Code
White	F6413046
Brown	F6413050
Lemon green	F6413054
Signal red	F6413058
Light beige	F6413047
Chocolate brown	F6413051
Jade	F6413055
Beige	F6413048
Black	F6413052
Royal blue	F6413056
Birch brown	F6413049
Platinum	F6413053
Corn yellow	F6413057

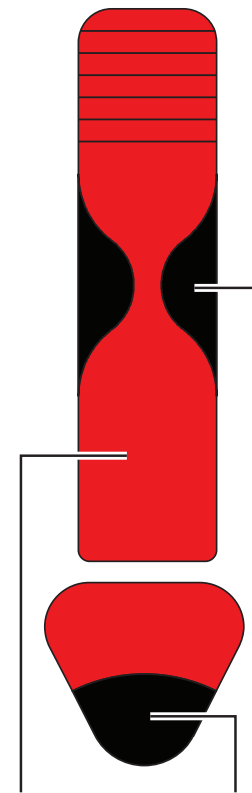

Main colour insert colour

EXAMPLES

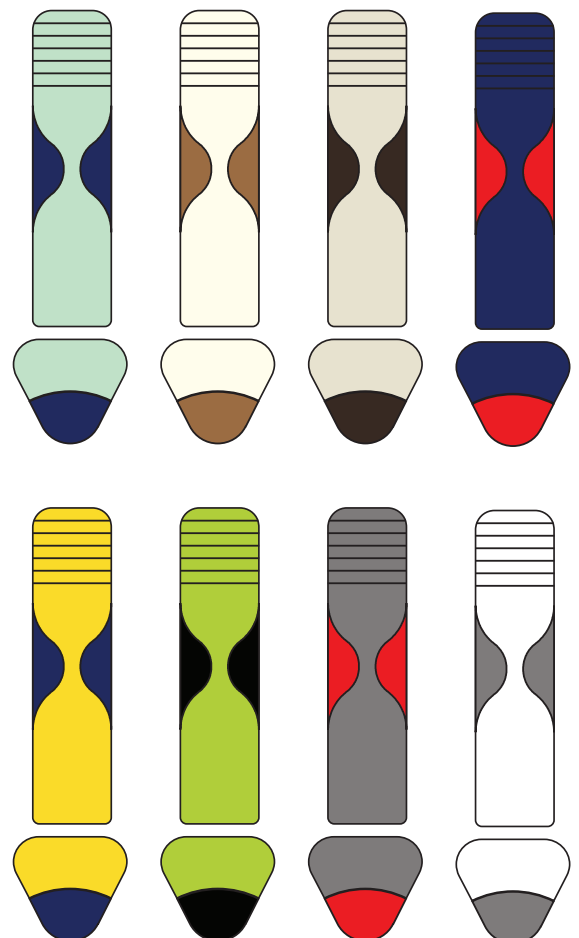

The standard colour for the EN-Dynamic equipment is 'royal blue' (code "F6413056" in the above colour scheme) for both the main and the contrasting colour. The standard colour of the fixation rolls is royal blue, however they can be ordered in black.

EN-DYNAMIC

STANDARD COLOURS

- Upholstery main colour: royal blue F6413056
- Upholstery insert colour: royal blue F6413056
- Fixationrolls: royal blue



OPTIONAL COLOURS, EXAMPLE 1

- Upholstery main colour: signal red F6413058
- Upholstery insert colour: black F6413052
- Fixationrolls: black



OPTIONAL COLOURS, EXAMPLE 2

- Upholstery main colour: chocolate brown F6413051
- Upholstery insert colour: brown F6413050
- Fixationrolls: black



OPTIONAL COLOURS, EXAMPLE 3

- Upholstery main colour: corn yellow F6413057
- Upholstery insert colour: royal blue F6413056
- Fixationrolls: royal blue



ORDER DATA AND TECHNICAL SPECIFICATIONS

Article nummer		Range	Dimensions	Weight
1455931	EN-Dynamic Total Hip	0-36x10 N (kgf)	110x115x170 cm	120 kg
1455925	EN-Dynamic Leg Extension	0-100x10 N (kgf)	140x90x140 cm	105 kg
1455932	EN-Dynamic Seated Leg Curl	0-70x10 N (kgf)	140x90x140 cm	105 kg
1455929	EN-Dynamic Abduction	0-60x10 N (kgf)	140x140x140 cm	85 kg
1455928	EN-Dynamic Adduction	0-60x10 N (kgf)	140x140x140 cm	85 kg
1455933	EN-Dynamic Seated Leg Press	0-200x10 N (kgf)	220x90x140 cm	105 kg
1455935	EN-Dynamic Squat	0-200x10 N (kgf)	250x80x150 cm	265 kg
1455921	EN-Dynamic Chest Press	0-120x10 N (kgf)	135x105x140 cm	100 kg
1455923	EN-Dynamic Pull Down	0-80x10 N (kgf)	140x120x180 cm	105 kg
1455927	EN-Dynamic Shoulder Press	0-60x10 N (kgf)	140x120x180 cm	135 kg
1455926	EN-Dynamic Rowing	0-100x10 N (kgf)	120x90x115 cm	100 kg
1455934	EN-Dynamic Triceps Dips	0-100x10 N (kgf)	130x72x140 cm	96 kg
1455930	EN-Dynamic Fly	0-65x10 N (kgf)	130x120x140 cm	100 kg
1455922	EN-Dynamic Low Back Trainer	0-80x10 N (kgf)	190x90x130 cm	120 kg
1455924	EN-Dynamic Abdominal Trainer	0-80x10 N (kgf)	160x90x140 cm	120 kg
1455936	EN-Dynamic Rotator	0-60x10 N (kgf)	135x95x108 cm	85 kg

OPTIONAL ACCESSORIES

- 1411800 EN-Dynamics installation kit
- 3444668 Compressor for EN-Dynamic
- 1455940 Range of Motion limiter EN-Dynamic Squat



1411800



3444668



1455940

OTHER PRODUCT THAT MIGHT INTEREST YOU

EN TRACK

The EN-Track concept gives you full control over the active rehabilitation process, ensuring optimum treatment at all times. Training schemes can easily be compiled using the training recommendations already pre-programmed in the EN-Track.

ORDERING DATA

- 1411801 EN-Track installation kit

Also see the separate product leaflets at WWW.ENRAF-NONIUS.COM/REHABILITATION

